

DATE: October 2019  
 SUBJECT: On-X Prosthetic Heart Valve MRI Information

**MR Conditional** 

The On-X Prosthetic Heart Valve, Conform-X Mitral Heart Valve Prosthesis; Size 25-33\* was determined to be MR-conditional according to the terminology specified in the American Society for Testing and Materials (ASTM) International, Designation: F2503-08. Standard Practice for Marking Medical Devices and Other Items for Safety in the Magnetic Resonance Environment. ASTM International, 100 Barr Harbor Drive, PO Box C700, West Conshohocken, Pennsylvania.

Non-clinical testing demonstrated that the On-X Conform-X Mitral Heart Valve Prosthesis; Size 25-33 is MR Conditional. A patient with this device can be scanned safely immediately after placement under the following conditions:

\*The MRI findings apply to this specific largest heart valve prosthesis and all other smaller sizes which are made from similar materials.

**Static Magnetic Field**

- Static magnetic field of 3-Tesla or less
- Maximum spatial gradient magnetic field of 720-Gauss/cm or less

**MRI-Related Heating**

In non-clinical testing, the On-X Prosthetic Heart Valve, Conform-X Mitral Heart Valve Prosthesis; Size 25-33 produced the following temperature rise during MRI performed for 15-min of scanning (i.e., per pulse sequence) in the 3-Tesla (3-Tesla/128-MHz, Excite, HDx, Software 14X.M5, General Electric Healthcare, Milwaukee, WI) MR system: Highest temperature change +1.6°C.

Therefore, the MRI-related heating experiments for the On-X Conform-X Mitral Heart Valve Prosthesis; Size 25-33 at 3-Tesla using a transmit/receive RF body coil at an MR system reported whole body averaged SAR of 2.9 -W/kg (i.e., associated with a calorimetry measured whole body averaged value of 2.7-W/kg) indicated that the greatest amount of heating that occurred in association with these specific conditions was equal to or less than +1.6°C.

**Artifact Information**

MR image quality may be compromised if the area of interest is in the exact same area or relatively close to the position of the On-X Conform-X Mitral Heart Valve Prosthesis; Size 25-33. Therefore, optimization of MR imaging parameters to compensate for the presence of this device may be necessary.

|                   |                       |                     |                       |                       |
|-------------------|-----------------------|---------------------|-----------------------|-----------------------|
| Pulse Sequence    | T1-SE                 | T1-SE               | GRE                   | GRE                   |
| Signal Void Size  | 1,090-mm <sup>2</sup> | 686-mm <sup>2</sup> | 1,478-mm <sup>2</sup> | 1,014-mm <sup>2</sup> |
| Plane Orientation | Parallel              | Perpendicular       | Parallel              | Perpendicular         |

\*The MRI findings apply to this specific largest heart valve prosthesis and all other smaller sizes which are made from similar materials.

For further information go to [www.MRIsafety.com](http://www.MRIsafety.com).

On-X Life Technologies, Inc. is a wholly owned subsidiary of CryoLife, Inc. The On-X Prosthetic Heart Valves are manufactured by On-X Life Technologies, Inc. CryoLife, the snowflake design, Life Restoring Technologies, and On-X are registered trademarks owned by CryoLife, Inc. or its subsidiaries. All other registered trademarks are owned by their respective owners. © 2019 CryoLife, Inc. All rights reserved. MLENG1132.001 (2019-10)